# Multi-Modality Fiducial Markers 52160

ENGLISH (EN)	1
БЪЛГАРСКИ (BG)	1
ČEŠTINA (CS)	2
DANSK (DA)	2
DEUTSCH (DE)	2
EESTI KEEL (ET)	3
ΕΛΛΗΝΙΚΑ (EL)	3
ESPAÑOL (ES)	3
FRANÇAIS (FR)	4
한국어 (KO)	4
HRVATSKI (HR)	4
ITALIANO (IT)	5
LATVIEŠU VALODĀ (LV)	5
LIETUVIŲ K. (LT)	5
MAGYAR (HU)	6
NEDERLANDS (NL)	6
日本語 (JA)	6
NORSK (NO)	7
POLSKI (PL)	7
PORTUGUÊS (PT)	7
ROMÂNĂ (RO)	8
РУССКИЙ (RU)	8
SLOVENČINA (SK)	8
SLOVENŠČINA (SL)	9
SUOMI (FI)	9
SVENSKA (SV)	9
TÜRKÇE (TR)	10
中文 (ZH)	10

## EN

### User Instructions:

### Intended purpose / intended use / indications for use:

For use as an artificial fiduciary marker on a patient's skin to facilitate localizing and documenting the precise superimposition of imaging data from CT, MRI, and other imaging sources.

### Contraindications and side effects:

No known contraindications or side effects.

### How to use the Disposable Marker:

1. Prep skin to remove excess oil and hair as necessary.
2. Peel off and discard protective backing.
3. Press the exposed adhesive of the marker firmly against the skin.
4. Repeat for each marker.
5. Recap tightly to prevent dry-out.

### Caution:

- To be used external to body only

### Disposal:

Dispose of the Multi-Modality Fiducial Markers in accordance with your hospital's regulations and local legislation.

### Symbols definition

 Manufacturer	 Use-by date	 Batch code	 Catalogue number
 Medical device	 Do not use if package is damaged	 Do not re-use	 Authorized representative in the European Community/European Union
 "Quantity of products in packaging"	 Date of manufacture	 U.S. federal law restricts this device to sale by or on the order of a physician	 Keep away from sunlight
 Keep dry	 Unique device identifier	 Consult instructions for use	 Importer

 The CE Label indicates that the product complies with the General Safety and Performance Requirements of European Regulation 2017/745, the Medical Device Regulation ("MDR"). Multi-Modality Fiducial Markers are a Class I product according to the rules established by the MDR.

For further information on the navigated use of the Multi-Modality Fiducial Markers please see the Brainlab user manual or consult Brainlab customer support.

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ILUMARK GmbH
Brainlab AG