IZI Medical

CE  
0123

EN  
2

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Bruksanvisning

## OSTEO-SITE® MIXING AND INJECTION SYSTEM

**CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).**

### DEVICE DESCRIPTION

The Osteo-site Mixing and Injection System is a kit which combines the Minimix Cement Mixer and the Duro-Ject Bone Cement Injector.

### INTENDED USE

The Minimix Cement Mixer has been designed for mixing cement. The Duro-Ject Bone Cement Injector has been designed for placing bone cement into surgical sites.

### CONTRAINDICATIONS

None known

### WARNINGS

- To avoid overfilling bone site and to stop the column of bone cement from advancing, user must turn plunger handle counterclockwise at least one full turn.
- If during injection, you feel an increase in pressure or see that no bone cement is being advanced out of the syringe, do not continue injection. Release the pressure by turning plunger handle counterclockwise at least one full turn and eliminate flow obstruction before continuing.

### PRECAUTIONS

- Follow all institutional protocols for the procedure.
- The product is intended for use by physicians trained and experienced in bone access and infusion techniques.
- Hold the Minimix Cement Mixer at the bottom while mixing to avoid transfer of heat from your hand to cement.
- Follow the bone cement manufacturer's instructions exactly.
- Monitor cement injection under live fluoroscopic visualization.
- Do not over-pressurize the system; connecting tube may rupture.

### INSTRUCTIONS FOR USE

1. Remove all components from the tray.
2. Follow images at the end of this document for mixer instructions.
3. Once cement is inside syringe barrel, connect the extension tube to the luer adapter, followed by the injector handle.
4. Begin injecting cement by turning the plunger handle in a clockwise direction.  
Note: One full turn of the handle delivers 0.20 cc of cement.
5. Advance cement through the connecting tube in small increments, until a small amount of cement is expelled, which will ensure the system is free of air.
6. Securely attach the connecting tube to the access needle.
7. Slowly turn the plunger handle, advancing the cement under live fluoroscopic visualization until an adequate fill has been achieved.
8. Stop the cement flow by turning plunger handle in a counterclockwise direction at least one full turn.  
Note: If it is necessary to reposition the needle within the bone, relieve pressure by turning the plunger handle counterclockwise at least one full turn before repositioning the needle.
9. Disconnect the connecting tube from the needle before removing the needle from the patient.  
Note: If an additional injection is required, a second barrel, o-ring assembly, connecting tube, and funnel have been provided.

### HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from package, inspect the product to ensure no damage has occurred.

### REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local IZI sales representative for information on available literature.

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IZI Medical Products, LLC
U.S.A.
ILUMARK GmbH