ILUMARK™ Navigation Marker - Snap
(REF. 2010)

ILUMARK GmbH
Hohenlindner Str. 11c
85622 Feldkirchen
Germany
Tel: +49 (0)89 907 795 98-0
Fax: +49 (0)89 907 795 98-9
Email: info@ilumark.com
www.ilumark.com

CASCINATION AG
Steigerhübelstrasse 3
3008 Bern, Bern
Switzerland

IZI Medical Products, LLC.
5 Easter Court, Suite J
Owings Mills, MD 21117
USA

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This product meets the essential requirements of Medical Device Directive 93/42 EEC

**User Instructions for
ILUMARK™ Navigation Marker - Snap**

Intended use
ILUMARK™ Navigation Markers are intended to be used as image guided surgery system accessory to aid in auto-registration and localization of rigid patient anatomical structure in either open or percutaneous image guided surgery procedures. It is indicated for any medical condition in which the use of an image guided surgery system may be considered to be appropriate.

ILUMARK™ Navigation Markers can be used in following procedures:

- Stereotactic brain surgery
- Endoscopic Sinus Surgery
- Pedicle Screw Placement
- Reconstructive orthopedic surgery
- Soft tissue treatment

⚠ Other applications are not considered and are within the responsibility of the user.

ⓘ ILUMARK™ Navigation Markers - Snap are packed in green blister packaging. Use these markers only on snap pins (see picture).

Compatibility

ILUMARK™ Navigation Markers - Snap are compatible with navigation systems:

- Orthokey® BLU-IGS System
- Medtronic® Stealthstation
- Aesculap® Orthopilot

Instruction for use

Take the ILUMARK™ Navigation Markers - Snap out of the inner sterile packaging with sterile, dry and clean gloves and gently place the hole of the sphere over the snap pin. Press the marker onto the pin till you hear an audible click sound. Verify that the ILUMARK™ Navigation Marker - Snap is firmly attached to the pin.

⚠ Do not use a ILUMARK™ Navigation Marker - Snap if it is not securely attached to the pin.

Only use clean and dry ILUMARK™ Navigation Marker - Snap. If the surface of a ILUMARK™ Navigation Marker - Snap is soiled or wet, this will result in inaccurate navigation. To ensure accurate navigation, wet or soiled ILUMARK™ Navigation Markers - Snap must either be cleaned, or replaced with new ILUMARK™ Navigation Markers - Snap. Only use a soft cloth moistened with sterile water to clean the surface of soiled ILUMARK™ Navigation Markers - Snap. Thoroughly dry ILUMARK™ Navigation Markers - Snap before use.

⚠ If ILUMARK™ Navigation Markers - Snap are used on vibrating instruments, verify that they are securely attached in regular intervals.

⚠ If the sterile packaging is damaged or sterility is in doubt, do not use the ILUMARK™ Navigation Markers - Snap, but dispose those spheres of.

ⓘ Patient treatment with ILUMARK™ Navigation Markers - Snap should only be performed by physicians experienced with Image Guided Surgery systems.

Handling

ILUMARK™ Navigation Markers - Snap are highly sensitive medical devices. They should be handled with care and not dropped. Do not use defective or deformed ILUMARK™ Navigation Markers - Snap.

Storage

Products must be stored in the original unopened packaging in a dry place. Storage temperature range must be considered.

⚠ Do not re-sterilize ILUMARK™ Navigation Markers - Snap!

⚠ Damages to the surface or geometry of the ILUMARK™ Navigation Markers - Snap are likely to happen during re-sterilization. Damages may not be visible to the human eye, but will result in inaccurate navigation which may lead to patient injuries.

For further information please consult ILUMARK™ customer support.

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