ILUMARK™ Navigation Marker Snap (REF. 1010-05)

ILUMARK GmbH
Hohenlindner Str. 11c
85622 Feldkirchen
Germany
Tel. +49(0)89 907 795 98-0
Fax. +49(0)89 907 795 98-9
E-Mail info@ilumark.com
www.ilumark.com

 **IZI Medical Products, LLC.**
5 Easter Court, Suite J
Owing Mills, MD 21117
USA

Do not use if package is damaged.
Bei beschädigter Verpackung nicht verwenden.
Ne pas utiliser en cas d'emballage endommagé.
Non utilizzare in caso di confezione danneggiata.
No utilizar si el envase está dañado.

Store in dark place.
Vor Sonnenlicht geschützt aufbewahren.
Conservar à l'abri de la lumière du soleil.
Proteggere dall'esposizione alla luce del sole.
Proteger de la luz del sol durante su almacenamiento.

Store in dry place.
Trocken aufbewahren.
Conservar dans un lieu sec.
Conservare in luogo asciutto.
Conservar en un lugar seco.

Consider instructions for use.
Bitte Gebrauchsanweisung beachten.
Consultez la notice d'utilisation.
Consultare le istruzioni per l'uso.
Consulte las instrucciones de utilización.

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Lot number
Chargennummer
N° du lot
Numero di lotto
Número de lote

Quantity
Quantität
Quantité
Quantità
Cantidad

User Instructions for ILUMARK™ Navigation Marker - Snap

Intended use

ILUMARK™ Navigation Markers Snap are retro-reflective, spherical markers that can be attached to instruments with snap pins. They enable accurate tracking of attached instruments using a Transcranial Magnetic Stimulation system.

- ▲ Other applications were not considered and are subject to the responsibility of the user.
- ❗ Only use ILUMARK Navigation Markers Snap on snap pins (see picture).

Instruction for use

Take the ILUMARK Navigation Markers Snap out of the packaging with dry and clean gloves and gently place the hole of the sphere over the snap pin. Press the marker onto the pin until you hear an audible click sound. Verify that the Navigation Marker - Snap is firmly attached to the pin.

- ▲ Do not use a ILUMARK Navigation Markers - Snap if it is not securely attached to the pin.

Only use clean and dry ILUMARK Navigation Marker - Snap. If the surface of a Marker is soiled or wet, navigation will be inaccurate. To ensure accurate navigation, wet or soiled Markers must either be cleaned or replaced with new Markers. Only use a soft cloth moistened with water to clean the surface of soiled Markers. Thoroughly dry ILUMARK Navigation Markers - Snap before use.

- ▲ If ILUMARK Navigation Markers Snap are used on vibrating instruments, regularly verify that they are securely attached.

- ❗ Only physicians experienced with Transcranial Magnetic Stimulation Systems should treat patients with ILUMARK Navigation Markers - Snap.

Handling

ILUMARK Navigation Markers - Snap are highly sensitive medical devices. They should be handled with care and not dropped. Do not use defective or deformed Markers.

Storage

Products must be stored in the original unopened packaging in a dry place.

- ▲ Do not sterilize Navigation Markers - Snap! Navigation Markers are supplied nonsterile.
- ▲ Use only ILUMARK-approved system components and accessories, unless otherwise specified.

For further information, please contact ILUMARK customer support.

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